

DENTAL STONE & INVESTMENT



NORITAKE SUPER ROCK

The flow of NORITAKE SUPER ROCK is superior to conventional products and its thixotropic properties facilitate handling. It easily fills and reproduces the finest details of the impression. Its high compatibility with all impression materials ensure casts with excellent surface texture. To facilitate trimming, this super-hard stone is specially formulated so that its wet compressive strength is not excessive; yet it is extremely hard when dry.

COLOR	BROWN , NEUTRAL GRAY
NET. WT.	3kg , 25 lbs (11.34kg) , 50 lbs (22.68kg)

PROPERTY

MIXING RATIO OF WATER (water/100g powder)	20ml
MIXING TIME	60sec.
SETTING TIME	10min.
SETTING EXPANSION (2 hours later)	0.08%
WET COMPRESSIVE STRENGTH (1 hour later)	66.7MPa
DRY COMPRESSIVE STRENGTH	121.6MPa



NORITAKE STONE

Many casts can be poured at one time because of the long working time (slow setting rate) of NORITAKE STONE. Regardless of the impression material used, the cast will have a smooth surface. Its relatively low setting expansion improves the accuracy the resultant cast.

COLOR	YELLOW , WHITE
NET. WT.	25 lbs (11.34kg) , 50 lbs (22.68kg)

PROPERTY

MIXING RATIO OF WATER (water/100g powder)	23ml
MIXING TIME	60sec.
SETTING TIME	11min.
SETTING EXPANSION (2 hours later)	0.23%
WET COMPRESSIVE STRENGTH (1 hour later)	62.7MPa
DRY COMPRESSIVE STRENGTH	112.8MPa



NORI-VEST NORI-VEST ALUMINA NORI-VEST ZIRCONIA

NORI-VEST series are refractory die material for porcelain inlays, onlays and all ceramic crowns.

NORI-VEST : for EX-3
NORI-VEST ALUMINA : for CERABIEN
NORI-VEST ZIRCONIA : for CZR

FEATURES

- It has a sufficient strength to withstand easily baking and build-up.
- NORI-VEST is easy to remove after the porcelain is baked, reducing the risk of breaking the Inlays.
- The mixed material flows easily, filling all the details in the impression.
- The fit is excellent.

NORI-VEST BIG SET

POWDER	30g×99packs
LIQUID	200ml×3bottles
SYRINGE	1 piece

NORI-VEST SET

POWDER	30g×33packs
LIQUID	200ml
SYRINGE	1 piece

NORI-VEST ALUMINA SET NORI-VEST ZIRCONIA SET

POWDER	30g×33packs
LIQUID	240ml
SYRINGE	1 piece