

## IS Fluoro



IS Fluoro, is the clear Internal Stain to mimic the fluorescence of natural tooth. Not only applying it after baking of Body and Enamel porcelains, but also applying to surface of Zirconia coping for PFZ (porcelain fused to zirconia) and to surface of opaque for PFM (porcelain fused to metal).

IS Fluoro can be mixed with porcelain powder to increase their fluorescence at your discretion.

### Under UV light \*1



None of porcelain on the Zirconia Framework (right)  
IS Fluoro on the Zirconia Framework (center) / Vertical section of the natural molar (left)

### Under natural light



\*1 / 365nm

### Internal Stain Lineups

White	Incisal Blue 1	Incisal Blue 2	Mamelon Orange 1	Mamelon Orange 2
Cervical 1	Cervical 2	Cervical 3	Earth Brown	Reddish Brown
Salmon Pink	Red	A+	B+	C+
D+	Bright	<b>IS Fluoro</b>	Gray	

## SS Fluoro



SS Fluoro, is the Shade Base Stain Modifier to increase the fluorescence. Not only mixing with Shade Base Stain porcelain as Modifier, but also applying zirconia frameworks directly.

### Shade Base Stain Modifier Lineups

A+	B+	C+	D+	Cervical Orange
Incisal Blue1	Incisal Blue2	Gray	Salmon Pink	Earth Brown
<b>Fluoro</b>				

SYMBOLS USED IN A LABEL	MANUFACTURER	USE BY	BATCH CODE
	CAUTION, CONSULT ACCOMPANYING DOCUMENTS. ATTENTION, SEE INSTRUCTIONS FOR USE.	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY	
Warning	If the patient is hypersensitive to Dental Porcelain or any of the other components, this medical product should not be used. Or it should be only used under the strict supervision of the patient's doctor/dentist.		
EU Authorized Representative	Name : EMERGO EUROPE	Address : Molenstraat 15, 2513 BH, The Hague, The Netherlands	

