


Super Porcelain **EX-3**

SPEED ENAMEL

EX-3 Speed Enamel is a new item specially developed for 2 layers build-up technique. Comparing with the conventional enamel, Speed Enamel has a beautiful opalescent effect. The esthetic restoration can be obtained easily with a simple application method. It is suitable to not only esthetic work but mass-production.

Start your simple & beautiful work with Speed Enamel NOW !

2 Layers Build-up Technique with Noritake Speed Enamel

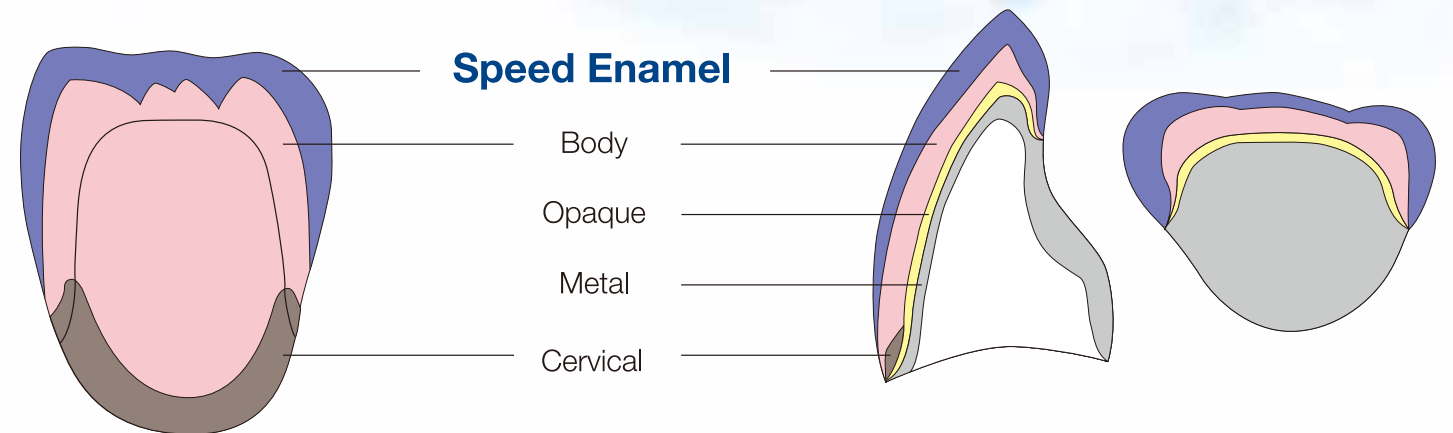

Clinical Case Application

1 After Paste Opaque application and bake

2 Pick Body and Speed Enamel

Body Application ▶

3

4 Cut-back

5 Mamelon Structure ▶

6 Apply Speed Enamel on Mamelon Structure

7 Apply Speed Enamel covering half of the surface

8 Cut-Back (lingual side)

9 Apply Speed Enamel on the lingual side

10 After bake and morphological correction

11 Apply Speed Enamel on the proximal area

12

Finish

el

External Stain

B+

	C ₁		C ₂		C ₃		C ₄		D ₂		D ₃		D ₄	
Paste Opaque	POC ₁	POnC ₁	POC ₂	POnC ₂	POC ₃	POnC ₃	POC ₄	POnC ₄	POD ₂	POnD ₂	POD ₃	POnD ₃	POD ₄	POnD ₄
Universal Paste Opaque	UPnC ₁		UPnC ₂		UPnC ₃		UPnC ₄		UPnD ₂		UPnD ₃		UPnD ₄	
Powder Opaque	C ₁ O	nC ₁ O	C ₂ O	nC ₂ O	C ₃ O	nC ₃ O	C ₄ O	nC ₄ O	D ₂ O	nD ₂ O	D ₃ O	nD ₃ O	D ₄ O	nD ₄ O
Margin	MC ₂ +MDL (1+1)		MC ₂	MC ₄ +MDL (1+1)		MC ₄	MD ₃ +MDL (1+1)		MD ₃	MD ₄				
Opaciously Body	OBC ₁		OBC ₂		OBC ₃		OBC ₄		OBD ₂		OBD ₃		OBD ₄	
Body	C ₁ B	nC ₁ B	C ₂ B	nC ₂ B	C ₃ B	nC ₃ B	C ₄ B	nC ₄ B	D ₂ B	nD ₂ B	D ₃ B	nD ₃ B	D ₄ B	nD ₄ B
Cervical	—		C ₂ B+CV-3 (2+1)		C ₃ B+CV-3 (1+1)		CV-3		D ₂ B+CV-4 (2+1)		D ₃ B+CV-4 (1+1)		CV-4	
Enamel	E ₂		E ₃		E ₃		E ₃		E ₂		E ₃		E ₃	
Speed Enamel	S₃		S₃		S₃		S₄		S₃		S₃		S₄	
External Stain	C+								D+					

Paste Opaque	PONP _{1.5}	PONP _{2.5}	PONW ₀	PONW _{0.5}	POEW ₀	POEW ₀	POEW	POEW
Universal Paste Opaque	UPNP _{1.5}	UPNP _{2.5}	UPNW ₀	UPNW _{0.5}	UPEW ₀	UPEW ₀	UPEW	UPEW
Powder Opaque	NP _{1.5} O	NP _{2.5} O	NW ₀ O	NW _{0.5} O	EW ₀ O	EW ₀ O	EWO	EWO
Margin	MNP _{1.5}	MNP _{2.5}	MNW ₀	MNW _{0.5}	MDL	MDL	MNW ₀ +MDL (1+1)	MNW ₀ +MDL (2+1)
Opaciously Body	OBNP _{1.5}	OBNP _{2.5}	—	—	—	—	—	—
Body	NP _{1.5} B	NP _{2.5} B	NW ₀ B	NW _{0.5} B	EW ₀ B	EW ₀ B	EW B	EWY B
Cervical	—	NP _{2.5} B+CV-1 (2+1)	—	—	—	—	—	—
Enamel	E ₂	E ₂	Silky E ₂	Silky E ₂	Silky E ₁	Silky E ₁	Silky E ₂	Silky E ₂
External Stain	A+		B+		A+		—	

SYMBOLS USED IN A LABEL

 MANUFACTURER	 USE BY	 LOT	 BATCH CODE	EU Authorized Representative Name: EMERGO EUROPE Address: Molenstraat 15, 2513 BH, The Hague, The Netherlands	 0120
 CAUTION, CONSULT ACCOMPANYING DOCUMENTS. ATTENTION, SEE INSTRUCTIONS FOR USE.	 EC REP	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY			

Noritake

NORITAKE DENTAL SUPPLY CO., LIMITED
300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi, 470-0293 Japan Phone +81-561-32-8953 Fax +81-561-32-8976

<http://www.noritake-dental.co.jp>