

Noritake

SUPER PORCELAIN EX-3 NEW COLOR KIT

NEW COLORS

The uses of New Colors :

Clear Margin

M Clear	Mix with margin porcelain to increase translucency
M Peach	A reddish orange for the cervical area
M Orange	A yellowish orange for the cervical area

Luster Porcelain

LT Natural	To increase incisal and proximal translucency for elders
LT Yellow	① To create a slight halo effect on the mamelons ② For achieving deep color in posterior occlusal surfaces

Opacious Body

OB White	To increase brightness,mix with the appropriate opacious body
OB Orange	To create a strong orange hue in the cervical or proximal area

Body

NW ₀ B	To create whitened shades For whiter shades
NW _{0.5} B	For whiter and slightly reddish shades

Paste Opaque

PONW ₀	For shade NW ₀
PONW _{0.5}	For shade NW _{0.5}

Clear Cervical

To create depth of color, without changing chroma

CCV-1	For bright shades like shades A ₁ to A ₃
CCV-2	For dark shades like A _{3.5} to A ₄
CCV-3	For reproducing surfaces exposed by receding gums
CCV-4	① Same as CCV-3, but with a more reddish hue ② For anterior lingual fossae

Enamel

Silky E ₁	① To express the calcification of natural enamel ② To reproduce a very bright bleached enamel
Silky E ₂	The recommended Enamel for NW ₀ and NW _{0.5}

Mamelon

Mamelon-1	A light mamelon color, also for crack lines
Mamelon-2	Strong mamelon color, may be mixed with Mamelon-1

Tissue

Tissue-1	An average tissue color
Tissue-2	A light tissue color
Tissue-3	A dark tissue color
Tissue-4	To be mixed with 1, 2 or 3 to increase value